

If the doctor or nurse has asked you to provide a sample to be sent to the hospital laboratory, please follow these steps:

(please note – not for urine Chlamydia sample)

Urine Monovette® User Guide

1 Remove stopper and keep for later use!
Attach plastic straw.

2 Insert the plastic straw into the container and fill the Urine Monovette® up to the bottom of the label.

3 To empty the plastic tube, hold the Urine Monovette® in an upright position and pull the plunger backwards to the bottom of the tube.

4 Remove the plastic straw break off the plunger and throw away.
Replace the stopper.

Waste

Technical modifications reserved

SARSTEDT SARSTEDT Ltd. · 68 Boston Rd, Leicester LE4 1AW · Tel.: 0116 235 9023 · Fax: 0116 236 6099
info@sarstedt.co.uk · www.sarstedt.com

